# Galvano-Welt

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## *The world of Electroforming*

*Electroforming | Hard-Gold-Plating | Friction Renewal*

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## *Gramm's electroforming technology*

*When the GAMMAT 12, the first electroforming unit that worked fully automatically, was launched on the market in 1989, this meant nothing else but the breakthrough of this accurate and biocompatible technique.*

*In the past 20 years, the dental technology and also the unit development have been in a state of flux. But one constant has persisted. **The name of the original is still the same: GAMMAT®.***

*Gramm is expert and leader of the world market in electroforming technology. The Gramm electroforming technique means more – more advantages for the dental technician as well as for the patient and for the dentist:*

## **More comfort**

- Very easy handling since all steps are indicated on the display
- Power and time settings are made automatically
- Automatic calculation of the amount of gold solution required

## **More quality**

- Homogeneous gold layer with a wall thickness of approx. 180–220 µm
- Exact fit
- Certified gold solution (CE 0483), no arsenic or other toxic substances added, cytotoxicity test passed

## **More efficiency**

- Reasonable investment costs
- No additional, expensive material requirements since most of the consumable materials are already available in the labs
- Integrated recycling programme to extract residual gold out of the gold solution

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# Electroforming – Biocompatibility and esthetics

Electroforming offers the possibility to produce self-bearing metal frameworks out of pure gold – without casting, sintering or high investments in new units – but in an electrochemical process.

**Bioesthetics** – this word, which has been coined by well-known material scientists and clinicians, expresses the main advantages of the electroforming technique. The Gramm Electroforming System (GES) delivers the best results regarding esthetics, material science, and reproducibility.

## Electroforming means:

- no allergic reactions to metals thanks to 24K gold
- accurate fitting precision thanks to direct deposition onto the duplicate model or the primary crown
- excellent esthetics
- only the electroforming technique provides a wall thickness of approx. 200µm without grinding, polishing, etc.
- outstanding economic efficiency due to very reasonable investment costs and low overheads
- to be used together with tried and tested fixing materials
- extremely low plaque deposition index which is even below that of natural dental enamel
- the gold margin may be burnished down
- clinically proven since 1961

## Indications

### Fixed restoration

- Crowns
- Inlays
- Onlays
- Bridges
- Splints
- Implants

### Removable restorations

- Bar-retained Restorations
- Telescopic Cases
- Implants
- Base Plates

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*Electroforming step by step at the example of a crown*

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*Electroforming step by step at the example of a telescopic crown*

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## Electroforming ...

### **... is evidence-based**

How much is a technique worth which has not been tried and tested?

The electroforming technique is established in dental labs all over the world. Its outstanding features are proven by numerous academic examinations and long-term studies. In addition, Gramm electroforming items distinguish by an unsurpassed shiny gold hue and extremely smooth surfaces.

### **... fits accurately**

Fitting precision is a decisive criterion for the durability of dental restorations. Thus, the risk of developing secondary caries is significantly minimized. Due to the specific technique by direct deposition of gold onto the duplicate model or the primary crown, this is the unbeatable advantage of the electroforming technology.

### **... means durability**

Biocompatibility and fitting precision offer best conditions for a favourable long-term prognosis. Satisfied patients – as well as the dentists – benefit thereof. In case of removable restorations, the fitting precision makes it easier for the patient to handle the denture – the friction is uniform and remains constant over a long period of time.

### **... is quality of life**

New teeth should be more than only a tooth replacement and allow the patient to smile, eat, and take part in life. Using Gramm's electroforming technique, the dental technician has almost ideal conditions to design teeth with natural esthetics. Delicate electroforming frameworks with their high stability provide a perfect basis, especially in implant prosthetics.

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*Golden Gate Bridge, San Francisco*

## Hard-Gold-Plating

### Friction renewal of worn telescopic cases

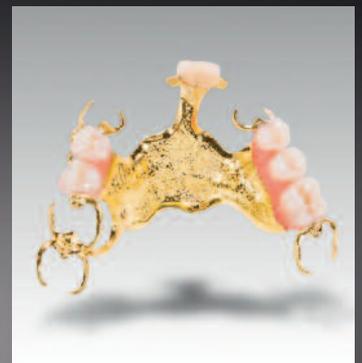
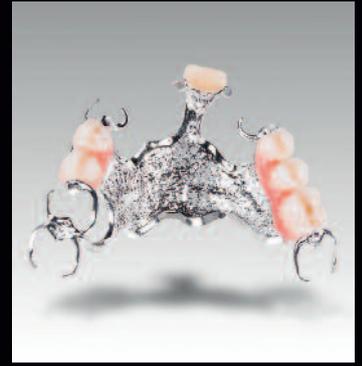
The esthetics of 24K gold as well as utmost biocompatibility are integrated within Gramm's gold-plating system. In order to make use of the entire range of indications, the unit GAMMAT® optimo2 can be extended by the gold-plating unit AU-SET.

#### Hard gold plating of Co-Cr frameworks

- Close, covering and securely adhering gold layer with an approx. thickness of 10 µm
- Cohesive gold layer preventing flake-off and corrosion
- Polish-resistant
- 24K electroforming gold, free of nickel and palladium
- Possibility to gold-plate model cast frameworks out of non-precious alloys even after a long wearing period

#### Friction renewal of worn telescopic cases

- Restoration of the adherence of worn telescopic prosthetics by selectively plating the inside of secondary crowns
- Saves time and money for the dental technician, for the dentist, and for the patient

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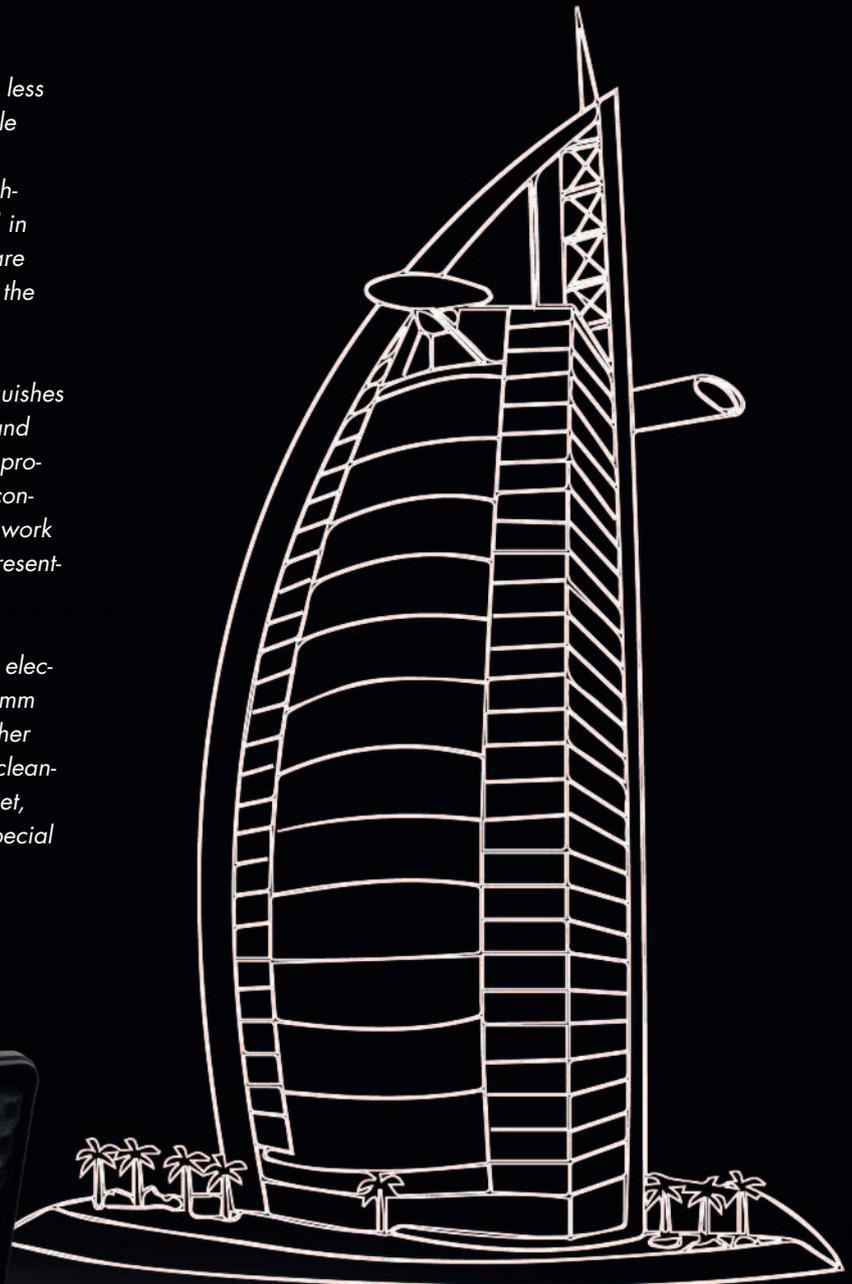
*Regarding electroforming, less is more, i.e. all consumable materials required for the Gramm electroforming technique, which are not used in your dental lab anyway, are available in the case with the starter kit. Just start – and achieve best results!*

*Even the accessory distinguishes by the passion for detail and the elaborate operational procedure. For example, all consumables required for the work preparation are orderly presented in the material stand.*

*As supplementation of the electroforming technique, Gramm also offers a number of other products for example the cleaningset CleanFix, a polish-set, a cutter-set as well as a special gold binder.*

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The GAMMAT® optimo2 offers:

- initialized via chip card providing easy software updates
- fully automated process run
- automatic contact verification prior to actual process
- complete review of all parameters prior to process start
- illuminated plating head with LED loading indication
- optiCLIC contacting system
- automatic power reduction in case of improper contacts
- extremely easy operation
- visualization with graphics on the colour display
- automatic calculation of the gold amount
- choice of the layer thickness
- revolutionary hydrodynamics for improved uniformity of the deposits, even in case of difficult shapes
- wide range of indication
- high material output
- recycling process
- control in case of power failure
- interface for PC to print a process protocol
- compact dimensions
- high economic efficiency
- very low power consumption
- friction renewal of worn telescopic cases
- nickel-free 24K hard gold plating

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## GAMMAT® optimo2 and AU-SET

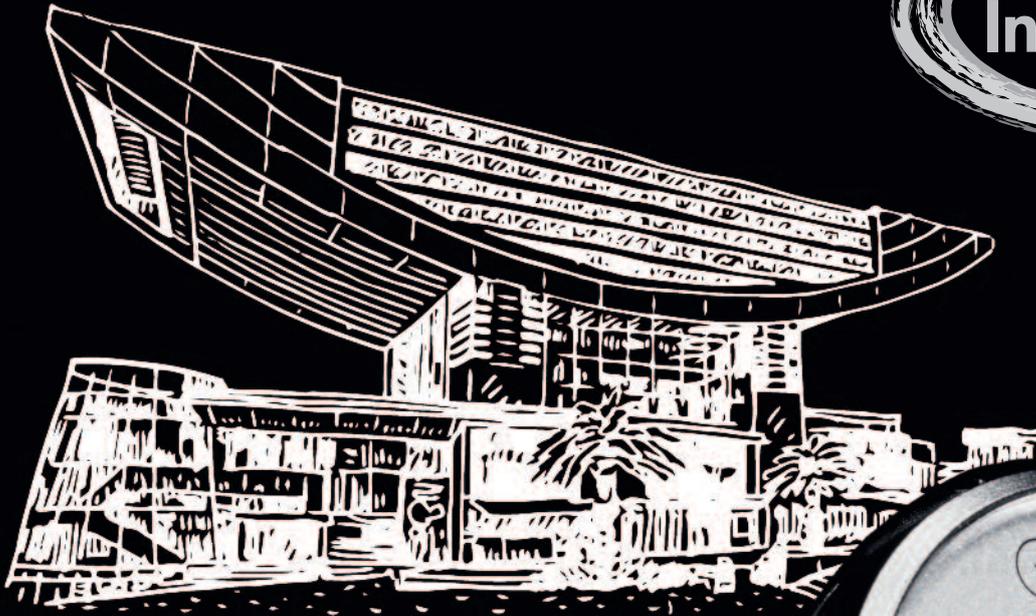
for Gramm Electroforming

Capacity	1 – 13 items
Power reception	200 W
Process run	5 hours
Layer thickness	200 or 260 µm
Requisite gold	0.4 – 10.0g Au in Gramm's gold solution ECOLYT SG200
Process Control	processor controlled, initialized via chip card, process fully automated
Recycling	integrated recycling process


for Gramm Gold-Hard-Plating and Friction Renewal

Capacity	depending on size of items
Process run	approx. 1 hour
Dimensions	(LxDxH) 90 x 31 x 20 cm
Total Weight	6.0 kg
Article No.	000.00.200 000.00.210

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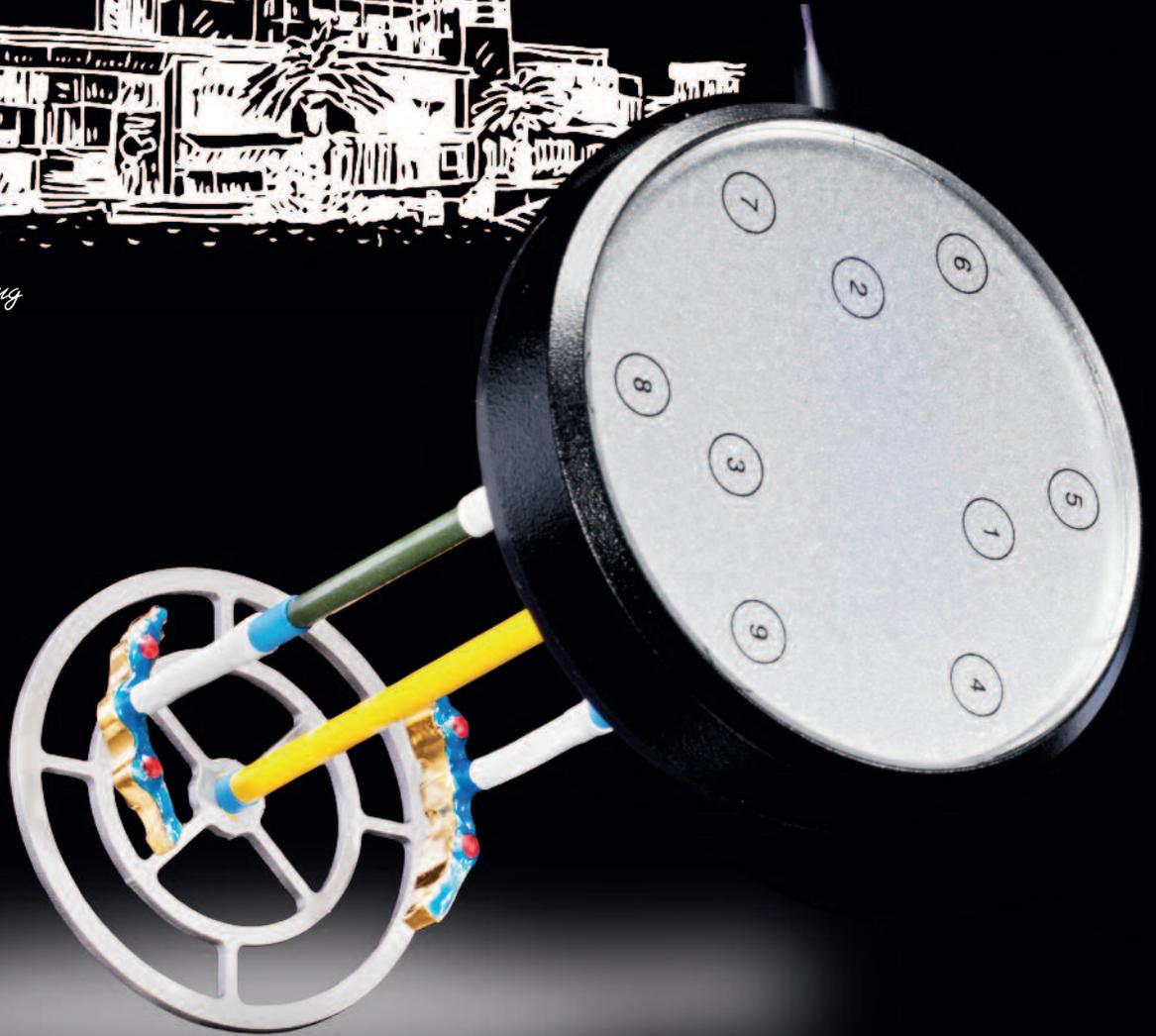**gramm**

Leader of the world market  
in dental electroforming

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**Gramm Technik GmbH