

Polishset (Art.-No. 910.00.096)

Content:

- 1 pc. diaPOLISH, diamond polishing paste for polishing with handpiece (Art.-No. 910.00.095)
- 1 pc. miniature buffering wheel (Art.-No. 910.00.097)
- 4 pcs. miniature brushes (Art.-No. 910.00.098)



Instructions for Use

Indication

diaPOLISH is a pre and high-lustre polishing with handpiece for:

- CAD/CAM zirconia ceramics, bars, telescopic and other parts
- Ceramic veneers **before** and **after** final firing
- Non-precious alloys, hard precious alloys



Technique

- 1 Run-in the polishing brushes or cotton buffing wheels by contacting a wooden plate or table surface at medium rotational speed.
- 2 Polish with miniature brush (Art.-No. 910.00.098) together with the paste. Apply paste sparsely, because it is very economical.
- 3 High-lustre polishing:
Use miniature buffering wheel (Art.-No. 910.00.097) – **without paste!**
Rotational speed of handpiece: 5.000–25.000 rpm
- 4 Cleaning:
Ultrasonic unit approx. 50°C, warm water with detergent or water steam cleaner.

Storage

Don't store above 25°C. Keep away from sunlight. Keep locked up and out of reach of children. Keep container closed.

Important

⚠ *Only for dental use by qualified personnel.*

These instructions result from tests made in our lab. The data may vary according to different conditions.



Gramm Technik GmbH
Dental Division
Einsteinstrasse 4
71254 Ditzingen-Heimerdingen
Phone +49 (0)7152 5009-60
Fax +49 (0)7152 5009-80
www.electroforming.de

revised: August 13th, 2013