

Cleaning Instruction for gold plated Galvano-Heads

The contacts of the Galvano-Head, the optiClic magnetic sticks and the anodes are gold plated. Before every electroforming process the contacts must be cleaned.

For cleaning only acetone applied with a cotton bud (Q-tip) should be used.

Do not use any other cleaning devices, they might damage the contacts!

In case you have any question, please call us 0049-7152-500960.



Gramm Technik GmbH
Geschäftsbereich Dental
Einsteinstraße 4
71254 Ditzingen-Heimerdingen
GERMANY